



Illustration only.  
Actual product may differ.

Photopolymer resin for the printing of dental soft tissue representations for non-oral use.

**Volume:** 1000 ml

**Curing wavelength:** 405nm

*General instructions:*

Shake bottle well before use.  
Do not refill used resin into bottle.  
When printing with used resin, filter, stir and shake well.  
Keep container tightly closed. Store locked up in a dark, dry place.

Use at room temperature.

## Material Safety

### PRECAUTIONS

Use only outdoors or in a well-ventilated area. Wear protective gloves/protective clothing/eye protection/face protection. Absorb spillage with suitable absorbent material. Transfer to a container for disposal.

### STORAGE

Store in tightly closed original container in a dry and cool place. Protect from freezing and direct sunlight. Keep away from heat, sparks and open flame. Keep away from human and animal food and drinks. Keep between 5 - 25 °C.

### DISPOSAL

Dispose of contents/container in accordance with national regulations. Avoid discharge to the aquatic environment. Do not discharge into drains, water courses or onto the ground.



**WARNING: Causes skin irritation.** Avoid contact with skin.  
WHEN CONTACTS SKIN: Wash contaminated skin thoroughly with plenty of soap and water.



**WARNING: Causes serious eye irritation.** Avoid contact with eyes.  
WHEN CONTACTS EYE: Rinse cautiously with water for several minutes. Remove contact lenses, if applicable. Continue rinsing. Seek medical advice/attention.



**WARNING: May cause respiratory irritation.** Avoid breathing vapor/spray.  
WHEN INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing.

MSDS available upon request.